



产品样册

Product Brochure

上海申安医疗器械厂
SHANGHAI SHENAN MEDICAL INSTRUMENT FACTORY



ENTERPRISE PROFILE

企业介绍

ShenAn Medical instrument

Shanghai ShenAn Medical Instrument Factory, with a history of over thirty years, is mainly engaged in producing medical instruments and laboratory apparatus. In recent years, in cooperating with the DGM in Switzerland, we keep on introducing international technology while combing our solid developing power. We are in protection of several proprietary intellectual property rights and are accredited with the multiple patents by National Patent Bureau. Both the quality of our products and the fame of our brand now have upgraded to a whole new level.

Our factory values innovation and intensifies management. It has been authorized with national Manufacture License of Special Equipments and Medical Instruments, and has acquired Quality Management System ISO9001 Certification, which promises the reliability and security of our products.

The two major product series of our factory are stainless steel pressure steam sterilizer and stainless steel distilling apparatus. At present, our products spreads all over the country and are exported to dozens of nations and districts, with our first class technology, completed types and specifications, classic and succinct style, as well as perfect and wide sales and after sales service network, we have built a good public image and have won the favor of a lot of customers.

Products of ShenAn are excellent in quality and preeminent in capability with perfect after service. We wish to provide all kinds of services to our customers both at home and abroad. Welcome to contact us!



上海申安医疗器械厂已经有三十多年的生产历史，产品涉及医疗器械，实验仪器领域。近年来，通过和瑞士 DGM 公司的技术合作，不断引进国外先进技术，结合自身雄厚的研发力量，产品具有自主知识产权，并获得国家专利局多项专利授权，使得产品品质持续提升，达到了一个全新的品牌平台。

本厂注重创新，强化管理，先后取得了国家颁发的特种设备生产许可证、医疗器械生产许可证，并通过了 ISO9001 质量管理体系认证，为产品的可靠性和安全性提供了更加有力的保障。

本厂专业生产的不锈钢压力蒸汽灭菌器和不锈钢蒸馏水器两大系列产品，目前，在国内同行业中技术领先，品种规格齐全，外观经典简洁，产品遍布国内所有省市并远销国外数十个国家和地区，具有完善和覆盖面广的销售和售后服务网络，在客户中树立了良好的形象，获得了广大用户的青睐。

申安产品以卓越的品质，超群的性能，完善的售后服务，竭诚为广大用户提供服务，热诚欢迎中外客商联系接洽、光临惠顾。

CONTENTS 目录

04 (医疗仪器)

Portable Pressure Steam Sterilizer
DSX 手提式压力蒸汽灭菌器

08 (医疗仪器)

Vertical Pressure Steam Sterilizer
LDZM 立式压力蒸汽灭菌器

10 (医疗仪器)

Vertical Pressure Steam Sterilizer
LDZF 立式压力蒸汽灭菌器

12 (医疗仪器)

Vertical Pressure Steam Sterilizer
LDZH 立式压力蒸汽灭菌器

14 (医疗仪器)

Horizontal Steam Sterilizer
WDZX 卧式蒸汽灭菌器

16 (实验仪器)

Portable Pressure Steam
Sterilizer (new)
DSX 手提式高压蒸汽灭菌器 (新品)

18 (实验仪器)

Portable Pressure Steam Sterilizer
DSX 手提式高压蒸汽灭菌器

20 (实验仪器)

Vertical Pressure Steam
Sterilizer (new)
LDZM 立式高压蒸汽灭菌器 (新品)

22 (实验仪器)

Vertical Pressure Steam
Sterilizer (new)
LDZF 立式高压蒸汽灭菌器 (新品)

24 (实验仪器)

Vertical Pressure Steam
Sterilizer (new)
LDZX 立式高压蒸汽灭菌器 (新品)

26 (实验仪器)

Vertical Pressure Steam Sterilizer
LDZH 立式高压蒸汽灭菌器

28 (实验仪器)

Horizontal Steam Sterilizer
WDZX 卧式高压蒸汽灭菌器

30 (实验仪器)

Stainless Steel Electric Water
Distiller (new)
DZS 不锈钢电热蒸馏水器 (新品)



DSX 系列

手提式 压力蒸汽灭菌器（医疗仪器）

Portable Pressure Steam
Sterilizer



型号 DSX-280B

容积 18 立升

Product Features

产品特点

实行微电脑智能控制系统：

采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：

采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免造成事故；

锅盖启闭装置：

采用移位式快开盖结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：

可预置固定程序对灭菌物品的灭菌；

记忆存储系统：

可存储上次灭菌程序，便于二次操作；

操控台动态：

数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：

锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、过流短路保护系统、漏电保护装置。

Intelligent microcomputer control system:

Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device:

The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening:

Using shift place type of quick-open cover to improve sealing and safety performance.

Preset program:

Can preset fixed program that sterilize solid body.

Memory storage system:

Store the last operation program so convenient next operation.

Digital display screen:

Digital display screen and working indicator light display running state.

Safety devices:

Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	DSX-280B
配置 Configuration	自控型 Semi-automatic control
材质 / 锅体壁厚 (mm) Material/Pot wall thickness(mm)	全不锈钢 / 2.0 Stainless steel / 2.0
容积 / 毛重 Volume / GW	18L / 15Kg
电压 / 功率 Voltage / Power	220V / 2Kw
包装尺寸 (长 × 宽 × 高 mm) Packing size (L X W X H mm)	405 x 405 x 615
容器容积 (直径 × 高度 mm) Sterilization chamber volume(D X H mm)	280 X 395
使用容积 (直径 × 高度mm) Use volume (D×H mm)	269×260
时间设定范围 Time setting range	0-999 min
额定工作压力 Rated working pressure	0.142MPa
灭菌温度设定范围 Sterilization temperature setting range	50~126℃



DSX 系列

手提式 压力蒸汽灭菌器（医疗仪器）

Portable Pressure Steam
Sterilizer



型号	DSX-280KB24	DSX-280KB30
容积	24 立升	30 立升

Product Features

产品特点

实行微电脑智能控制系统：

采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：

采用电子与机械互动的安全结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：

采用移位式快开盖结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：

可预置固定程序对灭菌物品的灭菌；

记忆存储系统：

可存储上次灭菌程序，便于二次操作；

操控台动态：

数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：

锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置。

Intelligent microcomputer control system:

Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device:

The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening:

Using shift place type of quickopen cover to improve sealing and safety performance.

Preset program:

Can preset fixed program that sterilize solid body.

Memory storage system:

Store the last operation program so convenient next operation.

Digital display screen:

Digital display screen and working indicator light display running state.

Safety devices:

Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	DSX-280KB24	DSX-280KB30
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control
材质 / 锅体壁厚 (mm) Material / Pot wall thickness(mm)	全不锈钢 / 2.0 Stainless steel / 2.0	全不锈钢 / 2.0 Stainless steel / 2.0
容积 / 毛重 Volume / GW	24L / 24Kg	30L / 32Kg
电压 / 功率 Voltage / Power	220V / 2Kw	220V / 2Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	500 X 500 X 800	500 X 500 X 900
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	370 X 430 X 730	370 X 430 X 820
灭菌室容积 (直径 x 高度 mm) Sterilization chamber volume (D X H mm)	280 X 495	280 X 585
使用容积 (直径 x 高度 mm) Use volume (D X H mm)	269 X 330	269 X 420
时间设定范围 Time setting range	0-999min	0-999min
额定工作压力 Rated working pressure	0.142MPa	0.142MPa
灭菌温度设定范围 Sterilization temperature setting range	50~126℃	50~126℃



LDZM 系列

立式 压力蒸汽灭菌器（医疗仪器）

Vertical Pressure Steam
Sterilizer



型号	LDZM-40KCS	LDZM-60KCS	LDZM-80KCS
	LDZM-40KCS-II	LDZM-60KCS-II	LDZM-80KCS-II
	LDZM-40KCS-III	LDZM-60KCS-III	LDZM-80KCS-III
容积	40 立升	60 立升	80 立升

Product Features

产品特点

实行微电脑智能控制系统：采用“LCD”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：采用拨杆式多连杆同步伸缩结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：可预置固定程序对固体的灭菌、干燥（适用于干燥配置）；

记忆存储系统：可存储上次灭菌程序，便于二次操作；

排汽控制：灭菌完成后，排气阀可按预先设定的灭菌时间倒计时结束后自动将蒸汽排至蒸汽收集箱，使灭菌过程无蒸汽外排（适用于蒸汽内排和干燥配置）；

校验接口：提供温度、压力检验接口，方便对锅内多点温度进行检测；

操控台动态：数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统；

Intelligent microcomputer control system: Using LCD digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Electric interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening: Using 8 rotary extendable stick to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize and dry solid body.

Memory storage system: Store the last operation program so convenient next operation.

Exhaust steam control: After sterilization, exhaust steam valve automatically work when sterilization time run out(apply to Auto exhaust and dry configuration).

Interface check: provide temperature and pressure check interface; convenient to check multiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Electric interlock device, overtemperature protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZM-40KCS LDZM-60KCS LDZM-80KCS	LDZM-40KCS-II LDZM-60KCS-II LDZM-80KCS-II	LDZM-40KCS-III LDZM-60KCS-III LDZM-80KCS-III
配置 Configuration	自控型 Semi-automatic control	全自动型 Full-automatic control	全自动型（带干燥） Drying
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	88/94/101kg(40/60/80L)	91/97/105kg(40/60/80L)	96/104/112kg(40/60/80L)
电压 / 功率 Voltage / Power	220V / 3.5Kw	220V / 3.5Kw	220V / 3.5Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	660 X 800 X 990 (40L)	660 X 800 X 1100 (60L)	660 X 800 X 1250 (80L)
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	560 X 630 X 880 (40L)	560 X 630 X 1000 (60L)	560 X 630 X 1150 (80L)
灭菌室容积 (直径 x 高度 mm) Sterilization chamber volume(D X H mm)	370 X 380 (40L)	370 X 560 (60L)	370 X 750 (80L)
网篮尺寸 (直径 x 高度 mm) Basket Size(D X H mm)	350 X 190 (40L)	350 X 360 (60L)	350 X 190 & 350 X 360 (80L)
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



LDZF 系列

立式 压力蒸汽灭菌器（医疗仪器）

Vertical Pressure Steam
Sterilizer



型号	LDZF-30KB	LDZF-50KB	LDZF-75KB
	LDZF-30KB-II	LDZF-50KB-II	LDZF-75KB-II
	LDZF-30KB-III	LDZF-50KB-III	LDZF-75KB-III
容积	30 立升	50 立升	75 立升

Product Features

产品特点

实行微电脑智能控制系统：采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：采用拨杆式多连杆同步伸缩结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：可预置固定程序针对固体的灭菌；

记忆存储系统：可存储上次灭菌程序，便于二次操作；

校验接口：提供温度、压力检验接口，方便对锅内多点温度进行检测；

操控台动态：数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统；

Intelligent microcomputer control system: Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilier's cover. Mack sure the operator's safety.

The structure of cover's opening: Using 8 rotary extendable stick to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize solid body.

Memory storage system: Store the last operation program so convenient next operation.

Interface check: provide temperature and pressure check interface; convenient to check multiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZF-30KB LDZF-50KB LDZF-75KB	LDZF-30KB-II LDZF-50KB-II LDZF-75KB-II	LDZF-30KB-III LDZF-50KB-III LDZF-75KB-III
配置 Configuration	自控型 Semi-automatic control	全自动型 Full-automatic control	全自动型（带干燥） Drying
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	79/86/93kg(30/50/75L)	81/89/95kg(30/50/75L)	81/89/95kg(30/50/75L)
电压 / 功率 Voltage / Power	220V / 3.5Kw	220V / 3.5Kw	220V / 3.5Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	660 X 610 X 970 (30L)	660 X 610 X 1120 (50L)	660 X 610 X 1270 (75L)
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	500 X 550 X 860 (30L)	500 X 550 X 1000 (50L)	500 X 550 X 1140 (75L)
容器容积 (直径 x 高度 mm) Sterilization chamber volume(D X H mm)	370 X 320 (30L)	370 X 500 (50L)	370 X 690 (75L)
网篮尺寸 (直径 x 高度 mm) Basket Size(D X H mm)	335 X 190 (30L)	335 X 360 (50L)	335 X 190 & 335 X 360 (75L)
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



LDZH 系列

立式 压力蒸汽灭菌器（医疗仪器）

Vertical Pressure Steam
Sterilizer



型号	LDZH-100KBS	LDZH-150KBS	LDZH-200KBS
容积	100 立升	150 立升	200 立升

Product Features

产品特点

实行微电脑智能控制系统：采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：采用手轮式旋压锁紧结构，使锅盖开启与锁紧安全可靠；

预置程序：可预置固定程序分别针对液体、固体的灭菌；

记忆存储系统：可存储上次灭菌程序，便于二次操作；

校验接口：提供温度、压力检验接口，方便对锅内多点温度进行检测；

操控台动态：数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统；

Intelligent microcomputer control system: Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Electric interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening: Using hand wheel rotary type cover to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize

Memory storage system: Store the last operation program so convenient next operation.

Interface check: provide temperature and pressure check interface; convenient to check mutiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZH-100KBS	LDZH-150KBS	LDZH-200KBS
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control	自控型 Semi-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	100L / 187Kg	150L / 200Kg	200L / 217Kg
电压 / 功率 Voltage / Power	380V / 6Kw	380V / 6Kw	380V / 6Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	760 X 800 X 1340 (100L)	760 X 800 X 1530 (150L)	760 X 800 X 1720 (200L)
容器容积 (直径 x 高度 mm) Sterilization chamber volume(D X H mm)	500 x 550	500 x 760	500 x 1000
使用容积 (直径 x 高度 mm) Use volume (D X H mm)	650 X 650 X 1200 (100L)	650 X 650 X 1400 (150L)	650 X 650 X 1700 (200L)
网篮尺寸 (直径 x 高度 mm) Basket Size(D X H mm)	465 X 230 & 2只 (100L)	465 X 230 & 3只 (150L)	465 X 230 & 4只 (200L)
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



WDZX 系列

卧式蒸汽灭菌器（医疗仪器）

Horizontal Steam
Sterilizer



型号	WDZX-120KC	WDZX-200KC	WDZX-300KC
容积	120 立升	200 立升	300 立升

Product Features

产品特点

实行微电脑智能控制系统：采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：采用拨杆式多连杆同步伸缩结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：可预置固定程序对固体的灭菌；

记忆存储系统：可存储上次灭菌程序，便于二次操作；

排汽控制：灭菌完成后，排气阀可按预先设定的灭菌时间倒计时结束后，数显屏有提示排放蒸汽；

校验接口：提供温度、压力检验接口，方便对锅内多点温度进行检测；

操控台动态：数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统；

Intelligent microcomputer control system: Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening: Using 8 rotary extendable stick to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize solid body.

Memory storage system: Store the last operation program so convenient next operation.

Exhaust steam control : After sterilization, exhaust steam valve should be opened (Digital display screen reminds exhaust steam) when sterilization time run out.

Interface check: provide temperature and pressure check interface; convenient to check multiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	WDZX-120KC	WDZX-200KC	WDZX-300KC
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control	自控型 Semi-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	120L / 225Kg	200L / 300Kg	300L / 333 Kg
电压 / 功率 Voltage / Power	380V / 2.5Kw X 3	380V / 3.0Kw X 3	380V / 4.0Kw X 3
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	1340 X 730 X 1700	1640 X 830 X 1800	1640 X 930 X 1900
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	1220 X 620 X 1540	1550 X 720 X 1640	1550 X 820 X 1740
容器容积 (直径 x 深度 mm) Sterilization chamber volume(D X H mm)	400X960	500 X 1120	600 X 1240
网篮尺寸(上长 x 上宽 x 下长 x 下宽 x 高度mm) Basket Size(Upper length X upper width X lower length X lower width X height mm)	380 X 340 X 380 X 225 X 140	480 X 440 X 480 X 250 X 195	500 X 540 X 500 X 300 X 230
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



DSX 系列

手提式 高压蒸汽灭菌器（实验仪器）

Portable Pressure Steam
Sterilizer

新品



型号 DSX-18L-I

容积 18 立升

Product Features

产品特点

实行微电脑智能控制系统：

采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：

采用电子与机械互动的安全结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：

采用移位式快开盖结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：

可预置固定程序对灭菌物品的灭菌；

记忆存储系统：

可存储上次灭菌程序，便于二次操作；

操控台动态：

数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：

锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置。

Intelligent microcomputer control system:

Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device:

The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening:

Using shift place type of quickopen cover to improve sealing and safety performance.

Preset program:

Can preset fixed program that sterilize solid body.

Memory storage system:

Store the last operation program so convenient next operation.

Digital display screen:

Digital display screen and working indicator light display running state.

Safety devices:

Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	DSX-18L-I
配置 Configuration	自控型 Semi-automatic control
材质 / 锅体壁厚 (mm) Material / Pot wall thickness(mm)	外壳：耐高温优质工程塑料，锅体：优质不锈钢 (304) 材料 / 2mm Shell: Temperature resistant high quality engineering plastics, pan: high quality stainless steel(304) material / 2mm
容积 / 毛重 Volume / GW	18L / 16Kg
电压 / 功率 Voltage / Power	220V / 2Kw
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	400 X 380 X 620
容器容积 (直径 x 高度 mm) Sterilization chamber volume (D X H mm)	280 X 395
使用容积 (直径 x 高度 mm) Use volume (D X H mm)	269 X 260
时间设定范围 Time setting range	0-999min
额定工作压力 Rated working pressure	0.142MPa
灭菌温度设定范围 Sterilization temperature setting range	50~126℃



DSX 系列

手提式 高压蒸汽灭菌器（实验仪器）

Portable Pressure Steam
Sterilizer



型号	DSX-24L-1	DSX-30L-1
容积	24 立升	30 立升

Product Features

产品特点

实行微电脑智能控制系统：

采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：

采用电子与机械互动的安全结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：

采用移位式快开盖结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：

可预置固定程序对灭菌物品的灭菌；

记忆存储系统：

可存储上次灭菌程序，便于二次操作；

操控台动态：

数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：

锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置。

Intelligent microcomputer control system:

Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device:

The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening:

Using shift place type of quickopen cover to improve sealing and safety performance.

Preset program:

Can preset fixed program that sterilize solid body.

Memory storage system:

Store the last operation program so convenient next operation.

Digital display screen:

Digital display screen and working indicator light display running state.

Safety devices:

Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	DSX-24L-I	DSX-30L-I
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control
材质 / 锅体壁厚 (mm) Material / Pot wall thickness(mm)	全不锈钢 / 2.0 Stainless steel / 2.0	全不锈钢 / 2.0 Stainless steel / 2.0
容积 / 毛重 Volume / GW	24L / 24Kg	30L / 32Kg
电压 / 功率 Voltage / Power	220V / 2Kw	220V / 2Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	500 X 500 X 800	500 X 500 X 900
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	370 X 430 X 730	370 X 430 X 820
灭菌室容积 (直径 x 高度 mm) Sterilization chamber volume (D X H mm)	280 X 495	280 X 585
使用容积 (直径 x 高度 mm) Use volume (D X H mm)	269 X 330	269 X 420
时间设定范围 Time setting range	0-999min	0-999min
额定工作压力 Rated working pressure	0.142MPa	0.142MPa
灭菌温度设定范围 Sterilization temperature setting range	50~126℃	50~126℃



LDZM 系列

立式 高压蒸汽灭菌器（实验仪器）

Vertical Pressure Steam
Sterilizer

新品



型号	LDZM-40L	LDZM-60L	LDZM-80L
	LDZM-40L-I	LDZM-60L-I	LDZM-80L-I
容积	40 立升	60 立升	80 立升

Product Features

产品特点

基本结构

- 1、外壳采用耐温优质工程塑料与不锈钢材料组合而成。
- 2、灭菌锅体采用优质不锈钢（304）材料精制而成。
- 3、锅盖启闭装置，采用拨杆式多连杆同步伸缩结构，使锅盖与锅体开启与闭合灵活轻巧，安全可靠。

操控与运行

操控台采用LCD液晶窗图文动态显示，功能设置均应用滚动式图文选择，整个灭菌行程实行微电脑图文显示及自动循环控制，灭菌结束（报警）后自动停机。

功能描述

- 1、具有一键制灭菌功能选择，可根据不同的灭菌物品快速明了地进行所需选择。
- 2、具有风冷式快速冷却装置，确保灭菌结束时对锅体快速降温，从而起到缩短开启锅盖时间。
- 3、具有灭菌物品的预热融化功能，对需要溶解的物品具有时间及温度的选择。
- 4、具有USB接口方便连接电脑监测灭菌过程及数据的收集。
- 5、具有灭菌时间预约功能，方便用户定时开机。
- 6、具有故障自动检测判断系统。
- 7、全自动型具有自动排放冷空气及灭菌结束自动排汽功能。
- 8、可增配打印功能，实时打印灭菌温度与灭菌时间。

安全保护装置

- 1、具有安全联锁装置，采用电子与机械互动的安全联锁结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免误操作而产生不安全。
- 2、具有机械式安全泄压阀和电控式过压保护装置的双套保护系统。
- 3、具有加热器防干燥保护功能和漏电保护系统。

I. Basic structure

1. The shell case is made of high temperature thermoplastic material with outstanding impact resistance and stainless steel.
2. The pot body of sterilizer is made of food grade stainless steel (304).
3. The lid opening and closing device adopts the lever type multi-link synchronous telescopic structure, which makes the lid and the pot body open and close flexible and light, safe and reliable.

II、 Control and operation

The console uses the LCD window graphic dynamic display, the function settings are all selected by scrolling graphic and text, the whole sterilization phase is monitored by micro-computer graphic display and automatic cycle control, and the automatic shutdown is initiated when sterilization phase ends (with an alarm).

III、 Function description

1. It has a one-button sterilization function selection, which can make a quick and clear choice based on different items.
2. With air-cooled chilling device to ensure that the pot body is quickly cooled at the end of sterilization, thus reducing the time to open the lid.
3. It has the function of preheating and melting of sterilized items and has time and temperature selection for the items to be dissolved.
4. A USB interface is available for the convenience of monitoring process with connection to a computer and of collecting data.
5. Sterilization reservation function is available so that users can start it at a set time.
6. Automatic fault detection and judgment system is in place.
7. The full-automatic control model has the function of discharging the low temperature air and exhausting steam when sterilization procedure ends.
8. Add print function, print the current temperature and sterilization time.

IV、 Safety devices

1. It has a safety interlocking device, structure of which is featured by electronic and mechanical interaction, thus ensuring that it automatically locks the lid when there is pressure in the pot and avoid any danger resulted from improper operations.
2. Double protection system with the mechanical safety relief valve and electronically controlled overvoltage protective device.
3. It has heater anti-drying protection function and leakage protection system.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZM-40L LDZM-60L LDZM-80L	LDZM-40L-I LDZM-60L-I LDZM-80L-I
配置 Configuration	自控型 Semi-automatic control	全自动型 Full-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	93/95.5/110Kg(40L/60L/80L)	97/99/113.5Kg(40L/60L/80L)
电压 / 功率 Voltage / Power	220V / 3.5Kw	220V / 3.5 Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	780×660×1160 (40L/60L)	780×660×1320 (80L)
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	570 X 690 X 1020 (40L) 570 X 690 X 1020 (60L)	570 X 690 X 1170 (80L)
容器容积 (直径 x 高度 mm) Sterilization chamber volume(D X H mm)	370 X 380 (40L) 370 X 560 (60L)	370 X 750 (80L)
网篮尺寸 (直径 x 高度 mm) Basket Size(D X H mm)	350 X 190 (40L) 350 X 360 (60L)	350 X 190 & 350 X 360 (80L)
时间设定范围 Time setting range	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C



LDZF 系列

立式 高压蒸汽灭菌器 (实验仪器)

Vertical Pressure Steam Sterilizer

新品



型号	LDZF-30L	LDZF-50L	LDZF-75L
	LDZF-30L-I	LDZF-50L-I	LDZF-75L-I
容积	30 立升	50 立升	75 立升

Product Features

产品特点

基本结构

- 1、外壳采用耐温优质工程塑料与不锈钢材料组合而成。
- 2、灭菌锅体采用优质不锈钢（304）材料精制而成。
- 3、锅盖启闭装置，采用拨杆式多连杆同步伸缩结构，使锅盖与锅体开启与闭合灵活轻巧，安全可靠。

操控与运行

操控台采用LCD液晶窗图文动态显示，功能设置均应用滚动式图文选择，整个灭菌行程实行微电脑图文显示及自动循环控制，灭菌结束（报警）后自动停机。

功能描述

- 1、具有一键制灭菌功能选择，可根据不同的灭菌物品快速明了地进行所需选择。
- 2、具有风冷式快速冷却装置，确保灭菌结束时对锅体快速降温，从而起到缩短开启锅盖时间。
- 3、具有灭菌物品的预热融化功能，对需要溶解的物品具有时间及温度的选择。
- 4、具有USB接口方便连接电脑监测灭菌过程及数据的收集。
- 5、具有灭菌时间预约功能，方便用户定时开机。
- 6、具有故障自动检测判断系统。
- 7、全自动型具有自动排放冷空气及灭菌结束自动排汽功能。
- 8、可增配打印功能，实时打印灭菌温度与灭菌时间。

安全保护装置

- 1、具有安全联锁装置，采用电子与机械互动的安全联锁结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免误操作而产生不安全。
- 2、具有机械式安全泄压阀和电控式过压保护装置的双套保护系统。
- 3、具有加热器防干燥保护功能和漏电保护系统。

I. Basic structure

1. The shell case is made of high temperature thermoplastic material with outstanding impact resistance and stainless steel.
2. The pot body of sterilizer is made of food grade stainless steel (304).
3. The lid opening and closing device adopts the lever type multi-link synchronous telescopic structure, which makes the lid and the pot body open and close flexible and light, safe and reliable.

II、 Control and operation

The console uses the LCD window graphic dynamic display, the function settings are all selected by scrolling graphic and text, the whole sterilization phase is monitored by micro-computer graphic display and automatic cycle control, and the automatic shutdown is initiated when sterilization phase ends (with an alarm).

III、 Function description

1. It has a one-button sterilization function selection, which can make a quick and clear choice based on different items.
2. With air-cooled chilling device to ensure that the pot body is quickly cooled at the end of sterilization, thus reducing the time to open the lid.
3. It has the function of preheating and melting of sterilized items and has time and temperature selection for the items to be dissolved.
4. A USB interface is available for the convenience of monitoring process with connection to a computer and of collecting data.
5. Sterilization reservation function is available so that users can start it at a set time.
6. Automatic fault detection and judgment system is in place.
7. The full-automatic control model has the function of discharging the low temperature air and exhausting steam when sterilization procedure ends.
8. Add print function, print the current temperature and sterilization time.

IV、 Safety devices

1. It has a safety interlocking device, structure of which is featured by electronic and mechanical interaction, thus ensuring that it automatically locks the lid when there is pressure in the pot and avoid any danger resulted from improper operations.
2. Double protection system with the mechanical safety relief valve and electronically controlled overvoltage protective device.
3. It has heater anti-drying protection function and leakage protection system.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZF-30L LDZF-50L LDZF-75L	LDZF-30L-I LDZF-50L-I LDZF-75L-I
配置 Configuration	自控型 Semi-automatic control	全自动型 Full-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	94/98/110Kg(30L/50L/75L)	97/101/113.5Kg(30L/50L/75L)
电压 / 功率 Voltage / Power	220V / 3.5Kw	220V / 3.5Kw
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	780×660×1160 (30L/50L)	780×660×1320 (75L)
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	570 X 630 X 970 (30L) 570 X 630 X 970 (50L)	570 X 630 X 1140 (75L)
容器容积 (直径 x 高度 mm) Sterilization chamber volume(D X H mm)	370 X 320 (30L) 370 X 500 (50L)	370 X 690 (75L)
网篮尺寸 (直径 x 高度 mm) Basket Size(D X H mm)	335 X 190 (30L) 335 X 360 (50L)	335 X 190 & 335 X 360 (75L)
时间设定范围 Time setting range	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C



LDZX 系列

立式 高压蒸汽灭菌器 (实验仪器)

Vertical Pressure Steam
Sterilizer

新品



型号	LDZX-30 L	LDZX-50 L	LDZX-75 L
	LDZX-30 L-I	LDZX-50 L-I	LDZX-75 L-I
容积	30 立升	50 立升	75 立升

Product Features

产品特点

基本结构

- 1、外壳采用耐温优质工程塑料与不锈钢材料组合而成。
- 2、灭菌锅体采用优质不锈钢（304）材料精制而成。
- 3、采用手轮式旋压密封结构，锅盖开启方式采用移位式开盖，使用方便，安全可靠。

操控与运行

操控台采用LCD液晶窗图文动态显示，功能设置均应用滚动式图文选择，整个灭菌行程实行微电脑图文显示及自动循环控制，灭菌结束（报警）后自动停机。

功能描述

- 1、具有一键制灭菌功能选择，可根据不同的灭菌物品快速明了地进行所需选择。
- 2、具有风冷式快速冷却装置，确保灭菌结束时对锅体快速降温，从而起到缩短开启锅盖时间。
- 3、具有灭菌物品的预热融化功能，对需要溶解的物品具有时间及温度的选择。
- 4、具有USB接口方便连接电脑监测灭菌过程及数据的收集。
- 5、具有灭菌时间预约功能，方便用户定时开机。
- 6、具有故障自动检测判断系统。
- 7、全自动型具有自动排放冷空气及灭菌结束自动排汽功能。
- 8、可增配打印功能，实时打印灭菌温度与灭菌时间。

安全保护装置

- 1、具有安全连锁装置，采用电子与机械互动的安全连锁结构，确保锅内有压力时连锁装置自动锁紧锅盖，避免误操作而产生不安全。
- 2、具有机械式安全泄压阀和电控式过压保护装置的双套保护系统。
- 3、具有加热器防干燥保护功能和漏电保护系统。

I. Basic structure

1. The shell case is made of high temperature thermoplastic material with outstanding impact resistance and stainless steel.
2. The pot body of sterilizer is made of food grade stainless steel (304).
3. hand-wheel type rotary sealing structure is adopted to make it easy to use, safe and reliable.

II. Control and operation

The console uses the LCD window graphic dynamic display, the function settings are all selected by scrolling graphic and text, the whole sterilization phase is monitored by micro-computer graphic display and automatic cycle control, and the automatic shutdown is initiated when sterilization phase ends (with an alarm).

III. Function description

1. It has a one-button sterilization function selection, which can make a quick and clear choice based on different items.
2. With air-cooled chilling device to ensure that the pot body is quickly cooled at the end of sterilization, thus reducing the time to open the lid.
3. It has the function of preheating and melting of sterilized items and has time and temperature selection for the items to be dissolved.
4. A USB interface is available for the convenience of monitoring process with connection to a computer and of collecting data.
5. Sterilization reservation function is available so that users can start it at a set time.
6. Automatic fault detection and judgment system is in place.
7. The full-automatic control model has the function of discharging the low temperature air and exhausting steam when sterilization procedure ends.
8. Add print function, print the current temperature and sterilization time.

IV. Safety devices

1. It has a safety interlocking device, structure of which is featured by electronic and mechanical interaction, thus ensuring that it automatically locks the lid when there is pressure in the pot and avoid any danger resulted from improper operations.
2. Double protection system with the mechanical safety relief valve and electronically controlled overvoltage protective device.
3. It has heater anti-drying protection function and leakage protection system.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZX-30L LDZX-50L LDZX-75L	LDZX-30L-I LDZX-50L-I LDZX-75L-I
配置 Configuration	自控型 Semi-automatic control	全自动型 Full-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	30L / 93Kg 50L / 96Kg	75L / 101Kg
电压 / 功率 Voltage / Power	220V / 3.5Kw	220V / 3.5Kw
产品尺寸 (长 × 宽 × 高 mm) Product size (L × W × H mm)	570 × 670 × 1240 (30L / 50L)	570 × 670 × 1340 (75L)
容器容积 (直径 × 高度 mm) Sterilization chamber volume (D × H mm)	370 × 400 (30L) 370 × 550 (50L)	370 × 700 (75L)
包装尺寸 (长 × 宽 × 高 mm) Packing size (L × W × H mm)	760 × 650 × 1270 (30L/50L)	760 × 650 × 1430 (75L)
网篮尺寸 (直径 × 高度 mm) Basket size (D × H mm)	335 × 190 (30L) 335 × 360 (50L)	335 × 190 × 3个 (75L)
时间设定范围 Time setting range	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.142MPa	0.142MPa
灭菌温度设定范围 Sterilization temperature setting range	50~126°C	50~126°C



LDZH 系列

立式 高压蒸汽灭菌器（实验仪器）

Portable Pressure Steam
Sterilizer



型号	LDZH-100L	LDZH-150L	LDZH-200L
容积	100 立升	150 立升	200 立升

Product Features

产品特点

实行微电脑智能控制系统:采用“LED”数显运行工作循环程序,灭菌结束(报警)后自动停机;

安全联锁装置:采用电子与机械互动的安全控制结构,确保锅内有压力时联锁装置自动锁紧锅盖,避免了误操作或使用不当而产生的不安全因素;

锅盖启闭装置:采用手轮式旋压锁紧结构,使锅盖开启与锁紧安全可靠;

预置程序:可预置固定程序分别针对液体、固体的灭菌;

记忆存储系统:可存储上次灭菌程序,便于二次操作;

校验接口:提供温度、压力检验接口,方便对锅内多点温度进行检测;

操控台动态:数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示;

安全配置:锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统;

打印功能(增配):实时打印灭菌阶段中的灭菌温度和灭菌时间。

Intelligent microcomputer control system: Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Electric interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening: Using hand wheel rotary type cover to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize

Memory storage system: Store the last operation program so convenient next operation.

Interface check: provide temperature and pressure check interface; convenient to check multiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Print function (add configuration): Print the current temperature and sterilization time.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	LDZH-100L	LDZH-150L	LDZH-200L
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control	自控型 Semi-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	100L / 188Kg	150L / 213Kg	200L / 238Kg
电压 / 功率 Voltage / Power	380V / 6Kw	380V / 6Kw	380V / 6Kw
包装尺寸(长 x 宽 x 高 mm) Packing size (L X W X H mm)	760 X 800 X 1340	760 X 800 X 1530	760 X 800 X 1720
产品尺寸(长 x 宽 x 高 mm) Product size (L X W X H mm)	650 X 650 X 1200	650 X 650 X 1400	650 X 650 X 1700
容器容积(直径 x 高度 mm) Sterilization chamber volume(D X H mm)	500 x 550	500 x 760	500 x 1000
网篮尺寸(直径 x 高度 mm) Basket Size(D X H mm)	465 X 230 & 2只	465 X 230 & 3只	465 X 230 & 4只
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



WDZX 系列

卧式 高压蒸汽灭菌器（实验仪器）

Horizontal Steam
Sterilizer



型号	WDZX-120L	WDZX-200L	WDZX-300L
容积	120 立升	200 立升	300 立升

Product Features

产品特点

实行微电脑智能控制系统：采用“LED”数显运行工作循环程序，灭菌结束（报警）后自动停机；

安全联锁装置：采用电子与机械互动的安全控制结构，确保锅内有压力时联锁装置自动锁紧锅盖，避免了误操作或使用不当而产生的不安全因素；

锅盖启闭装置：采用拨杆式多连杆同步伸缩结构，使锅盖开启与锁紧安全可靠、灵活轻巧；

预置程序：可预置固定程序对固体的灭菌；

记忆存储系统：可存储上次灭菌程序，便于二次操作；

排汽控制：灭菌完成后，排气阀可按预先设定的灭菌时间倒计时结束后，数显屏有提示排放蒸汽；

校验接口：提供温度、压力检验接口，方便对锅内多点温度进行检测；

操控台动态：数显屏与指示灯可显示设定的灭菌工作动态与运行的功能提示；

安全配置：锁盖联动装置、电控式超温过压保护系统、防干烧保护功能、机械式安全泄压阀、漏电保护装置、自动故障检测系统；

打印功能（增配）：实时打印灭菌阶段中的灭菌温度和灭菌时间。

Intelligent microcomputer control system: Using LED digital control system; After sterilization finished, sterilizer will automatic alarm and shut down.

Interlock device: The control system can real time supervision the sterilizer chamber's temperature and pressure. Only under the safety temperature and pressure can open the sterilizer's cover. Make sure the operator's safety.

The structure of cover's opening: Using 8 rotary extendable stick to improve sealing and safety performance.

Preset program: Can preset fixed program that sterilize solid body.

Memory storage system: Store the last operation program so convenient next operation.

Exhaust steam control : After sterilization, exhaust steam valve should be opened (Digital display screen reminds exhaust steam) when sterilization time run out.

Interface check: provide temperature and pressure check interface; convenient to check multiple position temperature measurement.

Digital display screen: Digital display screen and working indicator light display running state.

Safety devices: Interlock device, overtemperature and over pressure protecting system, burned protection, safety relief valve, current leakage protector and automatic fault detection.

Print function (add configuration) : Print the current temperature and sterilization time.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	WDZX-120L	WDZX-200L	WDZX-300L
配置 Configuration	智能半自动控制 Semi-automatic control	智能半自动控制 Semi-automatic control	智能半自动控制 Semi-automatic control
主体材质 The main material	不锈钢 Stainless steel	不锈钢 Stainless steel	不锈钢 Stainless steel
容积 / 毛重 Volume / GW	120L / 225Kg	200L / 300Kg	300L / 333Kg
电压 / 功率 Voltage / Power	380V / 2.5Kw X 3	380V / 3.0Kw X 3	380V / 4.0Kw X 3
包装尺寸 (长 x 宽 x 高 mm) Packing size (L X W X H mm)	1340 X 730 X 1700	1640 X 830 X 1800	1640 X 930 X 1900
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	1220 X 620 X 1540	1550 X 720 X 1640	1550 X 820 X 1740
容器容积 (直径 x 深度 mm) Sterilization chamber volume(D X H mm)	400 X 960	500 X 1120	600 X 1240
网篮尺寸(上长 x 上宽 x 下长 x 下宽 x 高度mm) Basket Size(Upper length X upper width X lower length X lower width X height mm)	380 X 340 X 380 X 225 X 140	480 X 440 X 480 X 250 X 195	500 X 540 X 500 X 300 X 230
时间设定范围 Time setting range	0-99 h	0-99 h	0-99 h
额定工作压力 Rated working pressure	0.217MPa	0.217MPa	0.217MPa
灭菌温度设定范围 Sterilization temperature setting range	50~134°C	50~134°C	50~134°C



DZS 系列

不锈钢电热蒸馏水器

StainlessSteel
ElectricWaterDistiller

新品



型号	DZS-5	DZS-10	DZS-20
出水量	5 升 / 时	10 升 / 时	20 升 / 时

Product Features

产品特点

- 水质符合2015版药典标准;
- 具有防烫外壳, 避免操作烫伤;
- 具有防缺水干烧功能;
- 具有节能减排的功效;
- 具有台式或挂壁两种安装模式。
- Water quality accord with Chinese pharmacopoeia (2015 edt).
- Shell material can prevent scalding and avoid Operation scald.
- Equipped with dry-fire protection device-can proof the dry-fire and heating up without water.
- Energy saving and emission reduction effecttion.
- Desk/hanging wall type installation mode.

Technical Parameter

技术参数

参数 / 型号 Parameter / Model	DZS-5	DZS-10	DZS-20
配置 Configuration	自控型 Semi-automatic control	自控型 Semi-automatic control	自控型 Semi-automatic control
主体材质 The main material	304不锈钢 Stainless steel	304不锈钢 Stainless steel	304不锈钢 Stainless steel
出水量 / 毛重 Out of water / GW	5L / h / ~9 Kg	10L / h / ~12 Kg	20L / h / ~15 Kg
电压 / 功率 Voltage / Power	220V / 3Kw X 1	380V / 2.7Kw X 3	380V / 2.5Kw X 6
产品尺寸 (长 x 宽 x 高 mm) Product size (L X W X H mm)	230 X 230 X 270	256 X 290 X 340	410 X 395 X 400
用水标准 Water category	生活用水 Regular water	生活用水 Regular water	生活用水 Regular water

01 Enterprise Qualification & Honor 企业资质与荣誉

- 医疗器械生产企业许可证【沪食药监械生产许 20000178 号】
- 特种设备制造许可证【TS2231005-202】
- 上海市消毒产品生产企业卫生许可证【沪卫消证字（2007）第 0005 号】
- ISO9001: 2015 国际质量管理体系认证
- “上海医疗器械名优产品”称号
- 上海市“五星级诚信创建企业”

02 Product Registration Number 医疗器械产品注册号

- （DSX 型）手提式压力蒸汽灭菌器【沪械注准 20172570653】
 - （LDZH 型）立式压力蒸汽灭菌器【沪械注准 20172570654】
 - （LDZM 型）立式压力蒸汽灭菌器【沪械注准 20202110432】
 - （LDZF 型）立式压力蒸汽灭菌器【沪械注准 20202110431】
 - （WDZX 型）卧式蒸汽灭菌器【沪械注准 20162570861】
-

AFTER SALES OUTLETS

售后网点

Perfect After Sales Service

上海申安医疗器械厂（总部）

上海市嘉定区安亭镇和静路2000号
TEL: 021-59578384 39586183
FAX: 021-59564552 39586180

辽宁省售后服务网点

辽宁省沈阳市于洪区怒江北街青山路阳光尚城4.1
期6号2.1.2室
TEL: 13940592288

江西省售后服务网点

江西省南昌市解放西路165号2栋1单元
TEL: 0791-8200849 13970844296
FAX: 0791-8200849

四川省(含重庆) 售后服务网点

成都市龙泉驿区悦来街97号武城大观城市花园二
期3栋1单元1号
TEL: 028-86932889 18883988881
FAX: 028-86932889

广西省售后服务网点

广西省南宁市西乡塘区秀灵路18号8栋31号
TEL: 13507714950 18978961612

北京、河北、天津、山东地区售后服务网点

河北省廊坊市安次区廊霸路139号
TEL: 15100686020

陕西省售后服务网点

陕西省西安市碑林区东木头32号
TEL: 029-87394948 13572270714
FAX: 029-87394948

安徽省售后服务网点

安徽省滁州市龙兴花园
TEL: 0550-3114076 18955005964
FAX: 0550-3114076



质量 服务 创新



上海市嘉定区安亭镇和静路2000号
2000Hejing Road Anting Town, Jiading District, Shanghai



021-59577853 59570238



021-59564552 39586180



www.shenan.com.cn



webmaster@shenan.com.cn saqy@shenan.com.cn